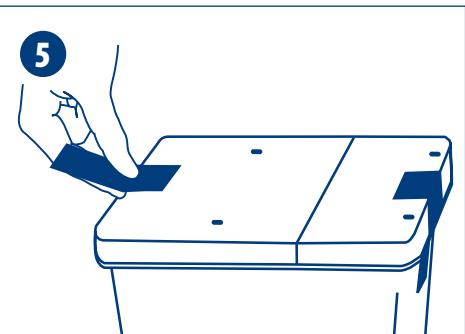
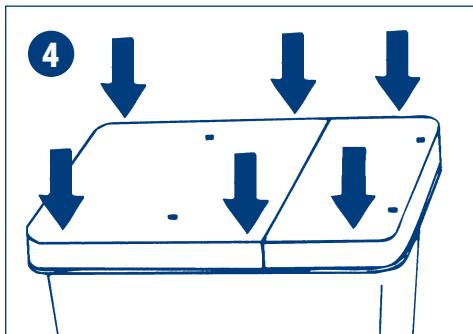
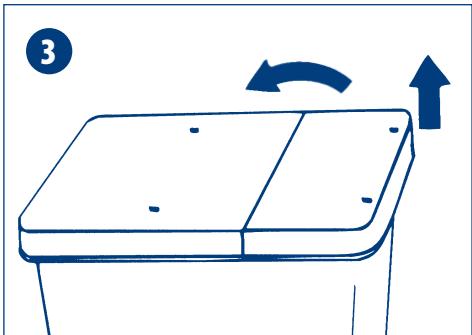
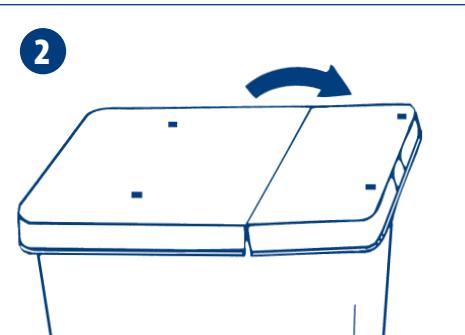
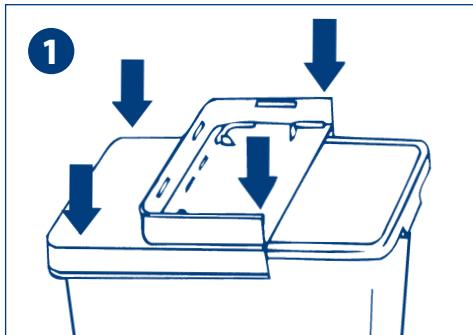
SharpSafety™

Pharmaceutical Waste Container

**Navette à objets tranchants
avec mécanisme de verrouillage
Estuche de traslado de objetos punzantes
con mecanismo de cierre
Vaivém de objectos cortantes com mecanismo de fecho**

- en** Assembly and Usage Instructions
- fr** Instructions d'Assemblage et d'Utilisation
- es** Instrucciones de Montaje y de Empleo
- pt** Instruções de Montagem e Utilização

en

Assembly and Usage Instructions

For SharpSafety Pharmaceutical Waste Container (Gasketed Hinged Lid)

1. Please check that the container has suffered no shipping damage before placing into use. Snap lid on container with hinged lid open. Listen for clicks. Place absorbent pad into container. Write date of first use and "Dispose of by" date on lid.
2. Hinged lid may be closed temporarily when not in use by lightly closing door.

To dispose of waste:

3. To use hinged lid – (A) Open and secure lid by locking lugs in place, or (B) Be sure lid is at rest in a fully open position. Use one hand to drop waste in container, then temporarily close hinged lid.

To final lock:

4. For final disposal – lock door by pressing down securely on edges of lid. Listen for clicks.

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en

Assembly and Usage Instructions

For SharpSafety Pharmaceutical Waste Container (Counterbalance Lid)

1. Snap lid on container by pressing down on all four sides. Listen for clicks. Important – make sure top is completely pushed down onto container. Check that the container has suffered no shipping damage before placing into use.
 2. Place container in wall enclosure or bracket, lift flap to close door and lock with key if it is a locking unit.
 3. Use one hand to place sharp in slot horizontally, making sure sharp falls into container. Lift to assure sharp drops! Do not dispose of more than one item at a time.
 4. Container is full if door does not return to the open position or the "full" indicator is visible.
 5. For final disposal – lock container before removing from enclosure or bracket. To lock, push flap all the way closed and snap both locking tabs down. Listen for clicks.
- Open enclosure or bracket and replace container.
- The use and contents of this container may dictate earlier disposal, but in no event should the container be more than three-quarters full. The container should be changed regardless of contents when objects can no longer be dropped freely, without resistance, into the container.

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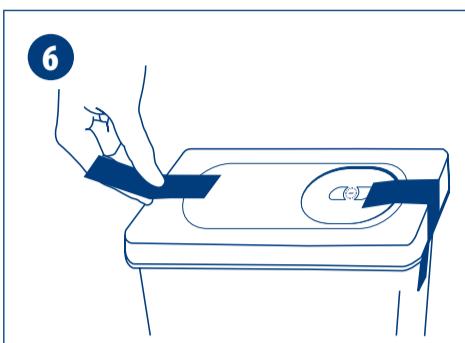
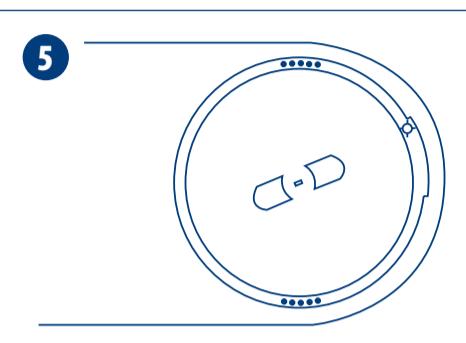
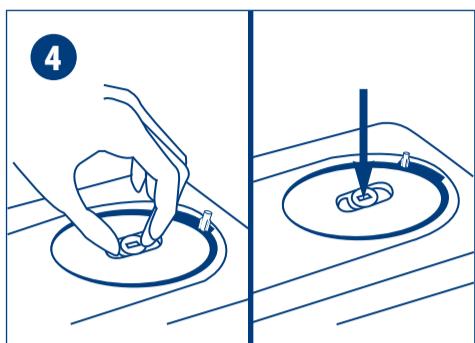
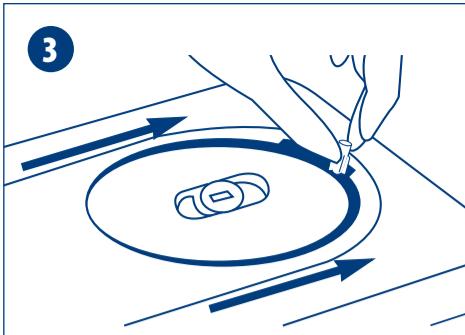
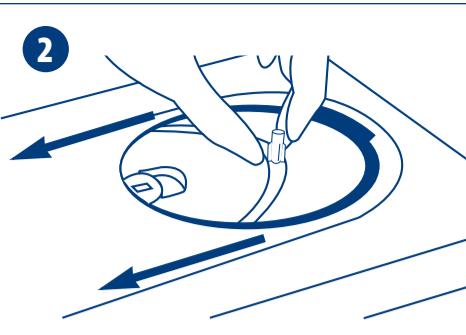
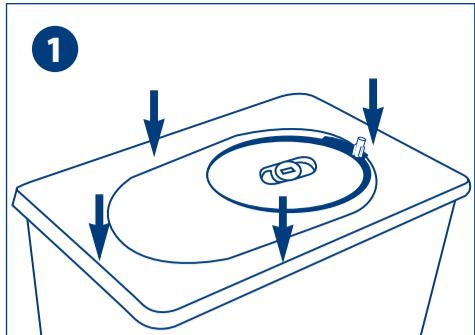
en

Assembly and Usage Instructions

For SharpSafety Pharmaceutical Waste Container (Gasketed Slide Lid)

1. Snap lid on container by pressing down on all sides. Listen for clicks. Please check that the container has suffered no shipping damage before placing into use.
2. To dispose of waste – with free hand on tab, slide door fully open. Move empty hand away from container. Drop waste in container. Drop one sharp in at a time.
3. To close door – after determining path is clear, slide tab until door is completely closed. Keep door closed when not in use.
4. For final lock – use center of circular door to turn counterclockwise until door drops down then press firmly in center of door to engage final lock. See step 5 for detailed final lock and disposal instructions.
5. Lid is locked when both sections of dots are clearly visible through the lid. Replace container immediately.
6. Attach tamper evident label to both sides of container. For incineration only.

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© 2011 Covidien.
COVIDIEN, COVIDIEN with logo and Covidien logo are U.S. and internationally registered trademarks of Covidien AG.
Other brands are trademarks of a Covidien company.
Covidien Inc., 15 Hampshire Street, Mansfield, MA 02048 USA.